


VENDLET Standard

QUICK GUIDE

THE USE

1. Elevate the bed to the right working height.
2. Double-click the  button to activate the VENDLET system. The VENDLET system produces an acoustic signal and the green diode over the power symbol turns on. When the hand control has been inactive for 90 seconds, the system is off again.
3. The patient is repositioned by operating the hand control.

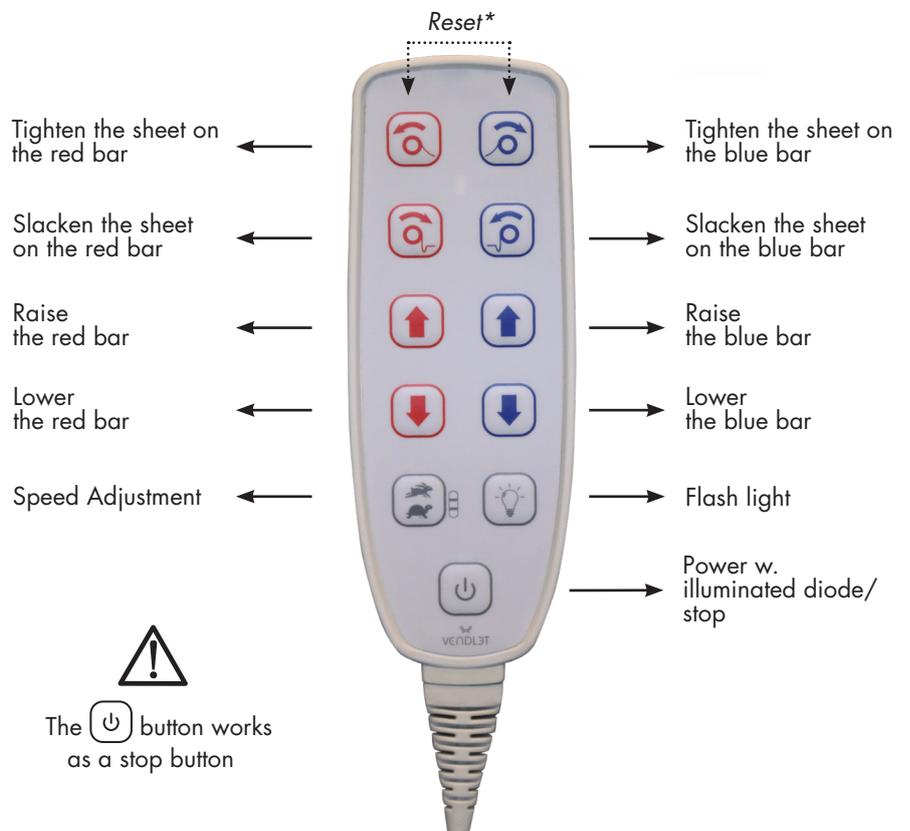

4. If it is estimated that the bedridden must use side rails, raise the bars. Slacken the sheet.
5. If it is estimated that the bedridden should not use side rails, lower the bars. Slacken the sheet.

Always, keep focus on the client when using VENDLET. 

* If the hand control is flashing, reset the system by simultaneously pressing the two top buttons on the hand control. You will hear 10 rhythmic beeps. Press and hold the buttons until the beeping stops.



VENDLET must be used and installed in accordance with the manual

Scan the QR code to watch instruction videos



VENDLET Standard

QUICK GUIDE



ATTACHING THE SLIDE SHEET

1. Fold the corner elastic straps around the mattress.
2. Check that the surface is smooth and even without folds. Make sure that slippery side is facing upwards.
3. Fasten the velcro straps around the edge of the mattress support platform. It is important that the straps are fastened around the edge of the platform under the mattress and not around the bed frame as this will prevent the back, thigh, and foot parts of the bed from being raised.



ATTACHING THE TURNING SHEET W. QUICK TAG

1. Place the turning sheet in the middle of the bed.
2. Press the edge strip of the turning sheet into the groove on one of the bars; begin at the head end of the bed and work your way down to the foot end.
3. The turning sheet must always be placed OVER the bars to prevent fingers, hair ect. of the patient from getting caught under the sheet. If the sheet is placed under the bars, the pulling power will be reduced and the motors will switch off when the turning sheet tightens.
4. Press the  or  button on the hand control and allow approx. two-thirds of the sheet to be pulled in by the bar.
5. Press the edge strip at the opposite edge of the turning sheet into the groove on the other bar.
6. Use this bar to pull in the turning sheet by pressing the  or  button until the sheet covers the mattress evenly with the red line at the centre of the mattress..



*VENDLET must be used
and installed in accordance
with the manual*

Scan the QR code to
watch videos on how
to attach the sheets

