

HEIGHT RESTRICTION


With some bed types problems arise when the VENDLET actuator collides with the bed wheels or foot brake when the bed height is lowered.

In several of these cases a clamping device can be mounted on the bed rod. This clamp ensures that the bed cannot be lowered so far as to collide with the VENDLET actuator, the bed wheels or brake.



The height restriction bracket can be mounted on all beds with the LINAK LA31 height adjustment motor, including:

- Invacare: SB750, SB750W, SB755 (85 and 90 cm), SB900 (2 units), SB 910 (2 units)
- Comfort System
- KR Opus

Mounting instructions

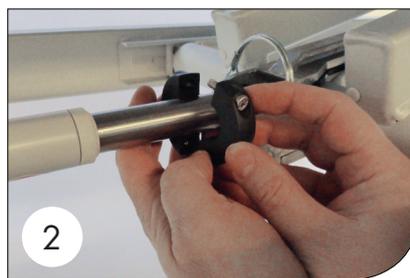

- 1.** Clamping device (product no. 6000035).
- 2.** Place the clamping device on the bed rod.
- 3.** Tighten the clamp so that it can be easily slid, without however sliding by itself.

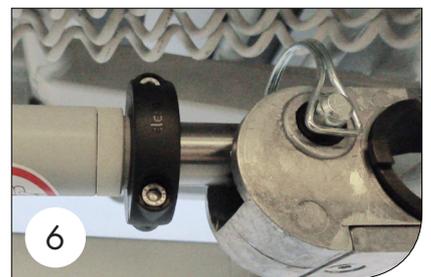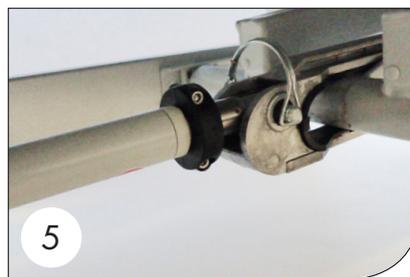

- 4.** The clamping device is now loosely mounted.
- 5.** Lower the bed to its lowest position. Raise the bed again to make space for working. Bring the clamping device 4 mm closer to the actuator housing. Tighten the clamping device well.
- 6.** Test the installation while making sure functions work. Run 5 times with the bed against the stop for further testing.